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Model: HY001

Product structure and composition EN:

This product is KN95 (HY001) protective mask of ship type, which is composed of mask body, nose clip and elastic belt. The mask body is protected by layers, consisting of PP non-woven fabric, Hot-air cotton, PP cotton type, Melt blown fabric and PP non-woven fabric.

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Product storage conditions EN:

Store in a room with relative humidity no more than 80%, no corrosive substances and good ventilation conditions.

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Use note EN:

1. Do not use in the environment with oxygen deficiency, toxic gas and excessive pollution.
2. When wearing the mask correctly, and smelling the odor or feeling that the breath is not smooth, the mask should be replaced immediately.
3. After using the mask, cut it into pieces, put in plastic bag and put it into the garbage can
4. This This protective face mask is suggested to be replaced every 4 hours. Not reusable.
5. Not suitable for children under 3 years old.
6. Not for use in operating rooms or invasive operations

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Product Validity EN:

This KN95 mask is valid for three years. Please use it with the validity period.

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- GB2626-2006 (for China export licence) KN95

Fitting Instructions EN:

1. Facing the face of the mask without nose clip, hold one ear band with both hands, so that the nose clip is located above the mask.
2. Put the mask over your chin.
3. After pulling the ear strap to the ear, adjust the ear strap to feel as comfortable as possible.
4. Place both hands and fingers in the middle of the metal nose clip, press inward while moving the fingertips along the nose clip to both sides, until the nose clip is fully pressed into the shape of the bridge of the nose.
5. Make sure the mask is covering your nose and mouth perfectly.

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